SRN: DE-MF-000021120

Basis UDI-DI: ++EWPDCMTLS

CopraTi-4 • CopraTi-5

Whitepeaks Dental Solutions GmbH • Weikenrott 17 • 46499 Hamminkeln • Germany • +49 281 2064580

	Gebrauchsanweisung	2
	Instructions for use	5
	Manuel d'utilisation	8
	Istruzioni per l'uso	11
	Instrucciones de uso	14
	Instrukcje użytkowania	17
	Instruktioner för användning	20
	Instruktioner til brug	23
	Bruksanvisning	26
	Käyttöohjeet	29
	Gebruiksaanwijzing	32
	Οδηγίες χρήσης	35
	Upute za uporabu	38
	Instrucțiuni de utilizare	41
	Naudojimo instrukcijos	44
	Návod k použití	47
	Návod na použitie	50
	Használati utasítás	53

SRN: DE-MF-000021120

Basis UDI-DI: ++EWPDCMTLS

CopraTi-4 • CopraTi-5

Whitepeaks Dental Solutions GmbH • Weikenrott 17 • 46499 Hamminkeln • Germany • +49 281 2064580

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Produkt/ Produkttyp

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Instructions for use

Product/ Product type

Titanium blanks for the fabrication of customised dental restorations

Material type

CopraTi-4: Pure titanium grade 4 • CopraTi-5: Titanium alloy grade 5 ELI

Class IIa medical device

Type 4 material in accordance with DIN EN ISO 22674.

Free from nickel, cadmium, beryllium and lead in accordance with DIN EN ISO 22674

Users

Professional users who fabricate customised dental restorations.

Target group

Patients of all ages and genders who require customised dental restorations

Indication / Intended use

The product is a medical device intended for the manufacture of dental prostheses intended for long-term use, which are partially inserted into the human body by means of a clinical procedure, but in this case are inserted into the teeth or attached to the tooth structure with suitable fixing material and are therefore classified as Class IIa.

CopraTi-4 (pure titanium grade 4)

Single crowns, medium bridges and bar constructions in the anterior and posterior region and superstructures

CopraTi-5 (titanium alloy grade 5 ELI)

Single crowns up to large bridges and bar constructions in the anterior and posterior region and superstructures

Contraindication

Do not use in case of proven hypersensitivity to one or more metals contained in the alloy

Veneering ceramics/ glaze

Titanium veneering ceramic/composite

Material properties/ Technical data

Material composition

Chemical composition in %	CopraTi-4 Pure titanium grade 4	CopraTi-5 Titanium alloy Grade 5 ELI
Ti	≥ 99%	≥ 90%
Al	---	5,5 - 6,5%
V	---	3,5 - 4,5%
Fe	≤ 0,50%	≤ 0,25%
C	≤ 0,08%	≤ 0,08%
N	≤ 0,05%	≤ 0,05%
O	≤ 0,40%	≤ 0,13%
H	≤ 0,0125%	≤ 0,012%

SRN: DE-MF-000021120

Basis UDI-DI: ++EWPDCMTLS

CopraTi-4 • CopraTi-5

Whitepeaks Dental Solutions GmbH • Weikenrott 17 • 46499 Hamminkeln • Germany • +49 281 2064580

Material properties/ Technical data

Chemical composition in %	CopraTi-4 Pure titanium grade 4	CopraTi-5 Titanium alloy Grade 5 ELI
Density	~ 4.51 g/cm ³	~ 4.43 g/cm ³
Vickers hardness	~ 180 - 210 HV10	~ 341 HV10
WAK (20°C-600°C)	~ 8,7 * 10 ⁻⁶ /K	~ 9,7 10 ⁻⁶ /K
Yield point	≥ 483 MPa	≥ 780 MPa
Tensile strength	≥ 550 MPa	≥ 860 MPa
Elongation at break	≥ 15%	≥ 10%
Hernia constriction	≥ 30%	≥ 15%

Processing

Milling

The desired restoration is milled from the selected blank.

Separate scaffolding

Cut out and grind frameworks and individual units using cross-cut tungsten carbide burs or cutting discs suitable for titanium. Always use the same rotating instruments for one metal to avoid contamination.

Cleaning

Finish the frameworks with a tungsten carbide bur suitable for titanium. To avoid the formation of bubbles when firing on the ceramic due to metal overlaps, remove the frameworks with the tungsten carbide bur in one direction only.

Then blast the surfaces with clean aluminium oxide (110µ) at 2 - 3 bar pressure and clean with superheated steam or methyl alcohol. Never use hydrofluoric acid.

Firing of the ceramic/composite

In principle, all commercially available ceramic bonding materials or composites suitable for titanium can be used. Please observe the processing instructions of the respective manufacturer and the CTE range specified therein for compatibility.

Safety instructions

Warning: Always wear respiratory protection (filter class FFP2), tight-fitting safety goggles, protective gloves and protective clothing and switch on extraction equipment. The dust generated during processing of this product can cause irritation to the skin/eyes/respiratory tract. Avoid inhalation/contact with skin, mouth and eyes. Do not eat or drink while working. Keep away from food and drink. Wash hands after handling. Remove contaminated clothing and protective equipment before entering eating areas. Keep away from sources of ignition. Do not smoke .

The resulting titanium chips can easily ignite when processed dry. Keep ignition sources away. Do not smoke. Avoid accumulation of swarf.

Storage

No special storage conditions. Store in the original packaging.

Waste disposal

Dispose of the product and packaging in accordance with local/ regional/ national/ international regulations. Do not dispose of with household waste. Do not allow to enter waterways, groundwater or sewage system.

SRN: DE-MF-000021120

Basis UDI-DI: ++EWPDCMTLS

CopraTi-4 • CopraTi-5

Whitepeaks Dental Solutions GmbH • Weikenrott 17 • 46499 Hamminkeln • Germany • +49 281 2064580

Hint

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

The Summary of Safety and Clinical Performance according to (EU) MDR 2017 / 745 Article 32 (SSCP) is available in the European database for medical devices (Eudamed) under the corresponding base UDI-DI as soon as the corresponding database module exists.

<https://ec.europa.eu/tools/eudamed>

Explanation of the labelling on the packaging

Confirmation: The product complies with the applicable European directives

Catalogue number/ item number

Batch designation

Unique product identification

Number of products in the packaging

Do not reuse

Observe the safety instructions in the operating instructions and instructions for use

www.wp-ifu.de/ti

Path to the electronic instructions for use

Instructions for use in paper form available within 7 days

Medical device

Date of manufacture

Manufacturer

Swiss authorised representative

Rx only

Caution: Federal law restricts this product to sale by or on the order of a licensed physician or dentist.

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