anaxDENT

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Instructions for use

APPLICATION

Blanks out of PMMA acrylic to produce temporary crowns and bridges in the anterior and posterior area.

COMPOSITION

Polymethyl methacrylate, Pigments.

HANDLING

anaxdent PMMA acrylic blanks are delivered in polymerized condition.

Vickers hardness (0,05 / 10): > 135
Tensile strength: > 75 Mpa
Bending strength: > 100 Mpa
E-modulus: > 2200 Mpa

PROCESSING

Please consider acrylic-specific settings when choosing the processing parameters. Verify after milling that the framework does not show material fractures as cracks. If a defect occurred, the framework must not be used as basis for a dental restoration. We recommend the use of eugenol-free temporary cements. For the manual processing of the blanks, cross-cut acrylic cutters suit best.

FACING AND RELINING

For the facing of frameworks as well as for the relining of the restoration, we recommend NEW OUTLINE temporary material which can be used directly in the mouth. An alternative would be to condition the framework with Bonding Fluid and to use anaxblend for the facing.

HANDLING AND STORAGE

This product may only be used by well-trained staff members. Please avoid dust development as well as dust deposition and make sure that the working area is properly ventilated by means of a suitable extraction system. Prevent vibrations and contamination. Keep in a dry place at room temperature and protect from direct sunlight.

MILLING

We remind you to consider the instructions of the units and materials used. Stick to the parameters given by the according manufacturer of the milling unit. In order to avoid overheating of the blank, choose milling parameters that are suitable for a PMMA program. (Please refer to the manufacturer of your milling unit for this.) If used in an AmannGirrbach milling machine, we recommend the following programs: Ceramil PMMA 71L for 20 mm blanks; Ceramil PMMA 71XS for 12 mm blanks.

1. OCCLUSAL THICKNESS

1 mm (min.)

2. CONNECTOR THICKNESS*

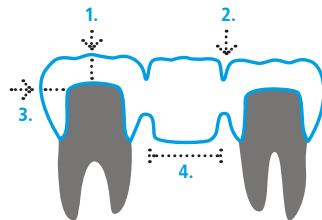
9 – 12 mm²

3. WALL THICKNESS

0,6 – 0,8 mm (min.)

4. SPAN WIDTH

15 mm (max.)

Please comply with the values given.

SAFETY WARNING

Our acrylic blanks are made and tested in accordance with the highest quality standards. This quality can only reach the patient if all instructions and recommendations are strictly kept. The improper use or noncompliance of the information given might affect the quality and, thus, can diminish the durability of the dental restoration. In general, this kind of restoration can remain in the mouth for a period of three to twelve months. Please note: Dusts produced during milling or grinding may irritate eyes, mucous membranes, or the skin.

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anax dent GmbH Olgastrasse 120a 70180 Stuttgart